All the reassurance you need

Tunstall

PART# 50100026 Ver 3.0

RTX3371 TELEHEALTH MONITOR

CE

INSTALLATION AND INSTRUCTIONS FOR USE	4	English
INSTALLATIONS- UND GEBRAUCHSANWEISUNG	10	Deutsch
INSTRUCTIONS D'INSTALLATION ET D'UTILISATION	16	Français
INSTRUCCIONES DE INSTALACIÓN Y FUNCIONAMIENTO	22	Español
ISTRUZIONI PER L'USO E L'INSTALLAZIONE	28	Italiano
INSTALLATIONS- OG BRUGERVEJLEDNING	34	Dansk
INSTALLATIE- EN GEBRUIKSINSTRUCTIES	40	Nederlands

English

INTENDED USE

Deutsch

Français

Español

Italiano

Dansk

Nederlands

The RTX3371 is for use in non-clinical settings (such as the home), as an accessory device that is intended to be a communication tool to enable healthcare providers to receive historical patient information. It is intended to be used in combination with a variety of external devices. The RTX3371 serves as the remote communication link between compatible external devices, and the compatible healthcare facility at another location. The healthcare facility could be at a disease management centre or with the healthcare/wellness provider or other out of hospital caregivers.

The purpose is to collect and transmit selected medical information (such as weight, blood pressure, blood glucose) using standard wireless technologies. The RTX3371 does not measure, interpret or make any decisions on the vital data that it conveys.

Contraindications, precautions and warnings

- This device is not intended for emergency calls, and may not be used for transmission or indication of any real-time alarms or time-critical data.
- All patient medical diagnosis and treatment are to be performed under the supervision and oversight of an appropriate healthcare professional.
- This device is not for use in systems, that substitute for medical care.
- This device is not intended for patients requiring direct medical supervision or emergency intervention.

Important information

- It is important that the cables be carefully arranged and secured to avoid unintended disconnection, or the possibility of tripping or blocking the path of a wheelchair or other moving objects.
- Be careful not to use a power socket that may be switched off unintentionally.
- Do not install or service the device, power adapter or cables during an electrical storm.
- Do not place the RTX3371 in extreme climatic conditions (excessive heat or humidity, in direct sunlight or where it may be exposed to large amounts of dust).
- Do not place beverages near the RTX3371 as spills may easily result in serious damage.
- Protect the device and cables from damage.
- Keep pets and small children away from the area where the RTX3371 and cables are located.
- Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.
- This equipment does not support emergency dialing and therefore cannot be used for this purpose.

External devices to be used with RTX3371

Scales:

- A&D Bluetooth Weight Scale “UC-321PBT”. Wireless Bluetooth connection.
- IEM Bluetooth Weight Scale (TC-100). Wireless Bluetooth connection.
- IEM Bluetooth Weight Scale “Libr-O-Graph”. Wireless Bluetooth connection.
- A&D Weight Scale “UC-321PL”. Cable connection.
- THT Bluetooth Weight Scale “SC-1” and “SC-2”. Wireless Bluetooth connection.
- THT Weight Scale “SC-1”. Cable connection.

INTENDED USE

English

Deutsch

Français

Español

Italiano

Dansk

Nederlands

Blood pressure monitors:

- A&D Bluetooth Blood Pressure Monitor “UA-767PBT”. Wireless Bluetooth connection.
- IEM Bluetooth Blood Pressure Monitor, “Stabil-O-Graph”. Wireless Bluetooth connection.
- A&D Serial Blood Pressure Monitor “UA-767PC”. Cable connection.

Blood glucose monitors:

- Roche “Accu-Chek Compact”, “Accu-Chek Compact Plus”, “Accu-Chek Aviva”, “Accu-Chek Performa” and “Accu-Chek Active” Blood Glucose meters. Infrared connection.
- Lifescan “OneTouch Ultra” and “OneTouch Ultra 2” Blood Glucose meters. Cable connection.
- Abbott/TheraSense “FreeStyle”, “FreeStyle Flash”, “FreeStyle Lite” and “FreeStyle Freedom Lite” Blood Glucose meters. Cable connection.
- Bayer “BREEZE®”, “BREEZE2®” and “CONTOUR®” (15- and 5-seconds version). Cable connection

Flowmeters:

- ViaSys AM1(+) Asthma Monitor. Wireless Bluetooth connection.
- MIR Spirotek Spirometer (also oximeter). Cable connection.

Oximeters:

- Nonin 4100BT Oximeter. Wireless Bluetooth connection.
- Nonin 9560 OnyxII BT Oximeter. Wireless Bluetooth connection.
- Nonin Ipod Oximeter. Cable connection.
- Mindray PM50 Oximeter. Cable connection.
- Mindray PM60 Oximeter. Infrared connection.

Coagulation monitor:

- Roche “CoaguChek” coagulation monitor. Infrared connection.

ECG recorders:

- Vitaphone 3100BT and 3300BT ECG monitors. Wireless Bluetooth connection.
- Corscience 3/6 and 12 BT ECG recorders. Wireless Bluetooth connection.

Warning

Do not connect other than the specified external devices to the RTX3371.

English

FUNCTIONALITY

Deutsch

Français

Español

Italiano

Dansk

Nederlands

The RTX3371 Telehealth Monitor is used for reception and communication of measurements taken by on-site external devices such as Blood Pressure Monitors, Personal Weight Scales, Blood Glucose Meters and other compatible devices: The communication will be performed using:

- Wireless data connection.
- Infrared data connection.
- Cable connection.

The display on the RTX3371 can be used to show questions that can be answered by pushing the selection buttons below the display. The current function of the right and left selection buttons will be stated in the display. If nothing is stated the button is disabled. A speaker is used to support the information shown in the display verbally. If a menu appears, where different options are listed the up and down buttons to the right of the display can be used to navigate in the list before selecting the appropriate choice. After the RTX3371 has received the measured values and answers, it generates a call and conveys the data to a remote facility via a GSM network. In some cases, the RTX3371 is programmed to deliver data at a specific time of the day. The delivery of data will automatically be repeated if the attempt to deliver data fails.

Cleaning and Maintenance

The RTX3371 may be cleaned with a soft, dry cloth or a cloth dampened with tap water and a mild detergent. A household glass cleaner on a soft, dry cloth may be used to clean off smudges. Never use any solvent, thinner, lighter fluid, alcohol, wet wipes or similar products on the RTX3371.

Note

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna of the other equipment.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from the one used for the receiver.
- Consult the manufacturer or an experienced radio/TV technician for help.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation. Modifications not expressly approved by TunstallHealthcare could void the user's authority to operate the equipment.

PARTS IDENTIFICATION

English

Deutsch



Français

Español

Italiano

Dansk

Nederlands

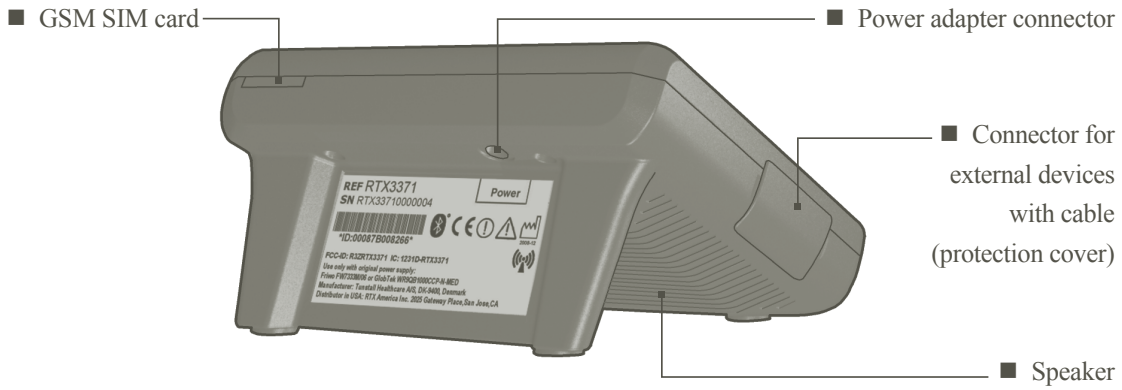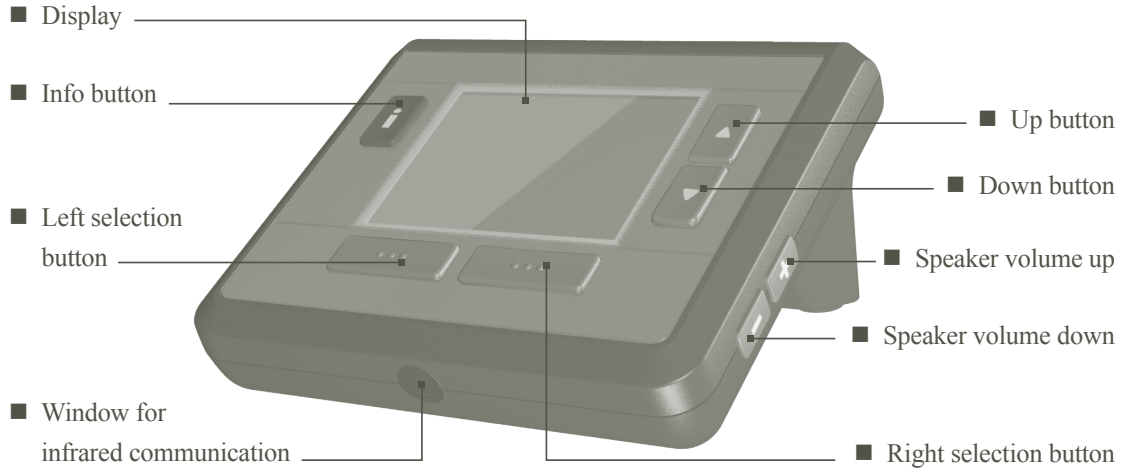English

INSTALLATION

Deutsch


Français

Español

Italiano

Dansk

Nederlands

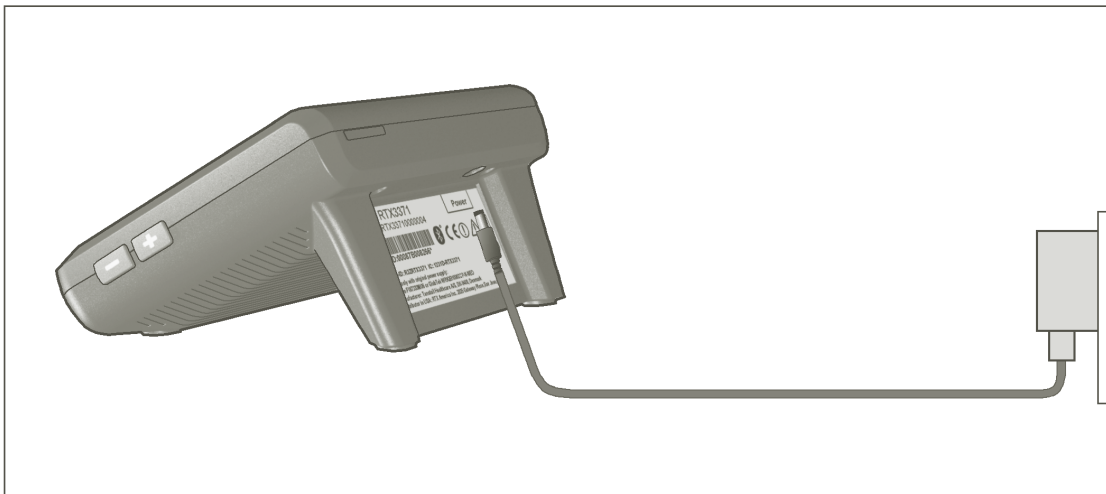Connect the power supply adapter to RTX3371 and to an electrical outlet.
The display of the RTX3371 will light up when power is connected.

Important

Use only the original power supply adapter Friwo FW7333M/06 or GlobTek WR9QB1000CCP-N-MED which comes with the RTX3371

The RTX3371 contains a small lithium battery. Do not replace.

Switching the RTX3371 off

If necessary the RTX3371 can be switched off by unplugging the power supply adapter from the electrical outlet.

Note

Use of mobile or portable phones or other RF emitting equipment near the RTX3371 may cause unexpected or adverse operation. Observe the operation of the RTX3371 when RF emitters are in the vicinity.

Warning

Use of other electrical equipment on or near the RTX3371 may cause interference. Verify normal operation of the RTX3371 in your installation before use.

TECHNICAL SPECIFICATIONS

English

Deutsch

Français

Español

Italiano

Dansk

Nederlands

- This device is for use at home.
- Power requirements: Mains: 100-240 VAC, 50-60Hz, 10W maximum (typical 2.4W), Class II, No applied part. Only use the original power supply adapter with the type designation FW7333M/06 or WR9QB1000CCP-N-MED.
- Power supply adapter: Equipment protected by double insulation or reinforced insulation.
CAUTION Risk of electrical shock. Dry location use only.
~ Alternating current and --- Direct current.
- Real time clock backup battery: Rechargeable Lithium battery
- Range of temperatures: Operating: +10°C to +40°C / +50 F to +104 F.
Storage: -10°C to +45°C / +14 F to +113 F.
Transportation: -20°C to +60°C / -4 F to +140 F.
- Relative humidity: Operating/Storage/Transport: 30% to 75% RH non-condensing.
- Barometric pressure: Operating/Storage/Transport: 700-1060 hPa.
- Dimensions (WxLxH): 145 x 125 x 75mm / 5.7 x 4.9 x 2.9 inches.
- Weight: 479 g / 17 oz including power supply adapter.
- Radio Transmitter: Bluetooth® compliance: Version 2.0.
Operating frequency: 2.402 to 2.480 GHz.
Output power: 1 to 100 mW / Bandwidth: <1MHz.
Antenna type: Internal.
Operational range: Up to 100 meters (indoor) depending of surroundings.
- Radio Transmitter GSM/GPRS:
GSM operating frequency: GSM850: 850 MHz, GSM 900: 900 MHz, DCS1800: 1800 MHz, PCS1900: 1900 MHz
GSM output power: Up to 2W
GPRS class: Class 10
- The internal antenna used for this mobile transmitter must provide a separation distance of at least 20 cm from the body and must not be co-located or operating in conjunction with any other antenna or transmitter.
- This device, including the power supply adapter, is certified to comply with IEC/UL60601-1.
- This device, including the power supply adapter, is tested according to IEC60601-1-2. More detailed information related to electromagnetic compatibility (EMC) is available upon request.
- Class-B digital apparatus complies with Canadian ICES-003.
- Data integrity and security: The privacy of data transmission with this device is secured by data encryption and authentication (SSL), and meets authorized standards under the HIPAA.
- Year and month of production of this device is printed underneath this symbol.
- Attention: See instructions for use (This manual).
- Only dispose the RTX3371 at an appropriate recycle facility for electronics. Ask Tunstall Healthcare for more information.
- The frequency band is not harmonized. Use of wireless Bluetooth technology can be restricted in some countries. Contact local authorities for more information.
- Tunstall Healthcare declares that this device is in conformance with the European Directive 93/42/ECC and 99/05/EC.

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RTX America, Inc. · 2025 Gateway Place · Suite 202 · San Jose · California 95110 · Phone: +1 (408) 441-8600